Dolomite® Legacy™

| | | |
|----|---|-----|
| en | Rollator User Manual | 3 |
| de | Rollator Gebrauchsanweisung | 23 |
| es | Rollator Manual del usuario | 45 |
| fr | Rollator Manuel d'utilisation | 67 |
| it | Rollator Manuale d'uso | 87 |
| nl | Rollator Gebruiksaanwijzing | 109 |
| pt | Andarilho Manual de utilização | 129 |

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for
future reference.

Dolomite

Yes, you can.®

©2020 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved
for future reference.

| | |
|---|-----------|
| 1 General | 4 |
| 1.1 Introduction | 4 |
| 1.2 Symbols in this Document | 4 |
| 1.3 Compliance | 4 |
| 1.4 Service Life | 5 |
| 1.5 Warranty Information | 5 |
| 1.6 Limitation of Liability | 5 |
| 2 Safety | 6 |
| 2.1 General Safety Information | 6 |
| 2.2 Labels and Symbols on the Product | 7 |
| 3 Product Overview | 9 |
| 3.1 Product Description | 9 |
| 3.2 Intended Use | 9 |
| 3.3 Indications for Use | 9 |
| 3.4 Scope of delivery | 9 |
| 3.5 Accessories | 9 |
| 4 Usage | 10 |
| 4.1 Before Use | 10 |
| 4.2 Folding and unfolding the walking aid | 10 |
| 4.3 Adjusting the Handle Height | 11 |
| 4.4 Walking | 11 |
| 4.5 Sitting Down on a Chair | 12 |
| 4.6 Rising to a Standing Position | 12 |
| 4.7 Sitting on the Walking Aid | 13 |
| 4.8 Using the brakes | 13 |
| 4.9 Adjusting the brakes | 13 |
| 4.10 Flip up seat plate | 14 |
| 4.11 Accessories | 14 |

| | |
|--|----|
| 4.11.1 Padded backrest strap (optional for most versions) | 15 |
| 4.11.2 Padded seat (optional for most versions) | 16 |
| 4.11.3 Basket | 17 |
| 4.11.4 Tray | 17 |

5 Maintenance **18**

| | |
|---|----|
| 5.1 General Maintenance Information | 18 |
| 5.2 Inspection Checklist | 18 |
| 5.3 Cleaning and Disinfection | 18 |
| 5.3.1 General Safety Information | 18 |
| 5.3.2 Cleaning Intervals | 19 |
| 5.3.3 Cleaning Instructions | 19 |

6 After Use **20**

| | |
|------------------------------|----|
| 6.1 Disposal | 20 |
| 6.2 Reconditioning | 20 |

7 Technical Data **21**

| | |
|--------------------------------------|----|
| 7.1 Dimensions and Weights | 21 |
| 7.2 Material | 22 |

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Serious Injury

- Incorrect use can lead to hazardous situations.
- Do not use on stairs or on escalators.
 - Do not stand on the product.
 - Be careful when walking on sloping or slippery ground or on uneven surfaces.
 - Avoid walking against obstacles, both forwards and backwards.
 - Do not let children play with the product.
 - Do not use the product to transport a person.
 - Make sure that no clothes, such as long skirts, can get stuck in the wheels.

CAUTION!

Risk of Injury

- Unbalanced or heavy load may lead to overturning.
- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
 - Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
 - The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
 - Refer to chapter 7 *Technical Data, page 21* for maximum user weight and load information.

CAUTION!
Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

CAUTION!
Risk of First Degree Burns

Parts of the product may heat up, when exposed to high temperatures (above 41° C).

- Be careful if the product is exposed to high temperatures, e.g. direct sunlight.

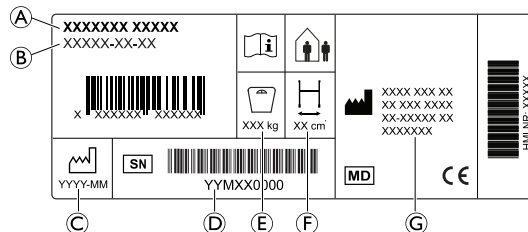
IMPORTANT!

There is a risk of damage due to wear, e.g. corrosion.

- Do not use the product under water, e.g. in pools.
- Do not use the product in hot areas like saunas.
- Do not store the product in wet or humid environments.

2.2 Labels and Symbols on the Product

Identification label

- Ⓐ Product name
- Ⓑ Part number
- Ⓒ Date of manufacture
- Ⓓ Serial number
- Ⓔ Maximum user weight
- Ⓕ Maximum width
- Ⓖ Manufacturer address

Symbols

| | | | |
|--|---------------------|--|------------------|
| | European Conformity | | Medical Device |
| | Manufacturer | | Read user manual |
| | Date of manufacture | | Serial number |

| | | | |
|---|----------------------|--|---------------------|
|  | Maximum width |  | Maximum user weight |
|  | Indoor / Outdoor use | | |

3 Product Overview

3.1 Product Description

The four-wheeled walking aid is equipped with a central seat that can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels. The walking aid is available in different sizes and can be equipped with a wide range of accessories.

3.2 Intended Use

The walking aid is intended to support during walking and may also serve as a resting seat.

The walking aid is suitable for indoor and outdoor use on even ground.

The intended user is an adult with a maximum user weight as stated on the product label and in chapter 7 *Technical Data*, page 21.

3.3 Indications for Use

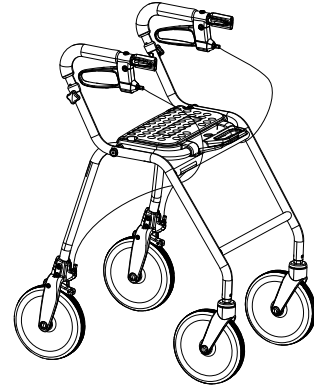
Indications

Persons with impaired muscle strength and/or balance, who need additional support whilst walking. The remaining walking and cognitive ability must be sufficient for safe use of a walking aid.

Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

3.4 Scope of delivery

- Walking aid, fully assembled.
- User manual

Accessories assembled to your walking aid, are depending on the configuration and might differ per region.

3.5 Accessories

Various accessories, like a basket / bag, are available for this product and can be found in the country-specific sales documents. Depending on the configuration accessories may be pre-assembled.

4 Usage

4.1 Before Use

CAUTION!

Risk of Injury or Damage to Property

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.

Invacare presume a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

4.2 Folding and unfolding the walking aid

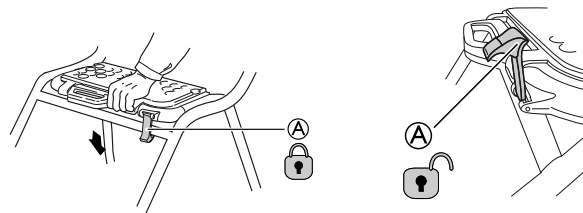
WARNING!

Risk of injury

- Never lift the rollator by the folding handle.

Unfolding to operating position

1. Hold the walking aid with the push handles and make sure that all four wheels are stable on the ground.
2. Press both push handles downwards. Make sure that the seat plate is folded completely down (see picture).
3. Push down the safety latch Ⓐ in the locked position.

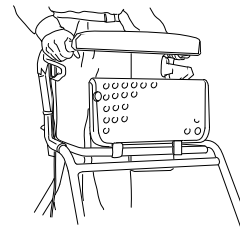
WARNING!

- Before using the product, always check that the safety latch Ⓐ is in locked position indicated by a clear audible “Click”.

Flip up the seat plate


The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

! **IMPORTANT!**

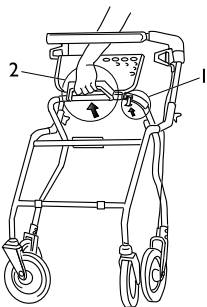The walking aid must be fully unfolded and locked with the folding handle before using the product.
– The folding handle locks when a clicking sounds is heard.

Folding

1. Lift up/loosen the safety latch.
2. Lift the handles in order to fold the walking aid into folded position.

4.3 Adjusting the Handle Height**WARNING!**
Risk of falling

- Always ensure that the hand knobs are tightened securely to prevent the handle from unintended lowering.
- Do not set the handles above their highest allowed position, indicated by the marking on the tubes.

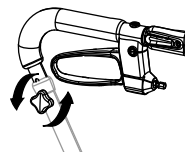
If the user is not supposed to adjust the height of the handles, e.g. for safety reasons, the hand knobs can be replaced by locking screws (available as accessory).


Recommended height: The handles should be positioned at wrist height when standing straight with relaxed arms. Therapist may suggest different heights depending on specific needs of the client.


1.

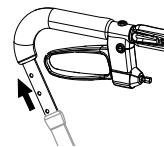2.

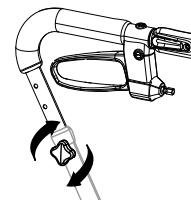
3.

1. Turn the hand knob counterclockwise and remove it. Remove the screw.
2. Adjust the handle to suitable height.
3. Insert the screw and the hand knob. Turn the hand knob clockwise to tighten.
4. Repeat the procedure on the other side.


4.4 Walking**WARNING!**
Risk of falling

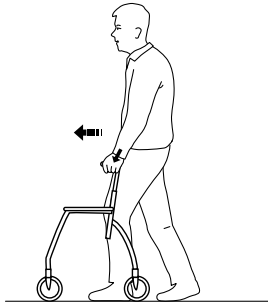- Be careful if the wheels stop against an obstacle, e.g. a kerb.

1. Place the walking aid slightly in front of you and take a step towards it while using the hand grips for support.

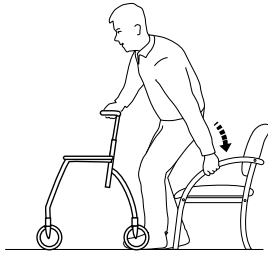4.5 Sitting Down on a Chair

1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Lock both hand brakes of the walking aid.
3. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.

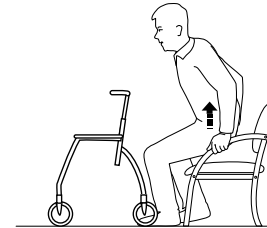
4.6 Rising to a Standing Position

WARNING!

Risk of tipping

- Do not use the walking aid to pull yourself up from a sitting position.

1. Place the walking aid in front of you and lock both hand brakes.
2. Use the armrests of the chair as support and rise carefully to a standing position.
3. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

4.7 Sitting on the Walking Aid

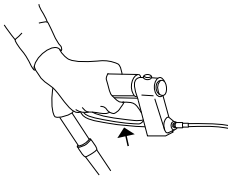
WARNING!

Risk of Falling

- Use the walking aid for stationary seating only.
- Before sitting down, lock both brakes in parking position and make sure the walking aid stands stable.
- Always sit on the walking aid from the rear side. Do not sit down from the front side.
- Do not sit on the walking aid on sloping ground.
- Do not maneuver the walking aid with your feet while sitting on it.

4.8 Using the brakes

Using the hand brakes


1. Pull both brakes upwards to brake.

Using the parking brake

IMPORTANT!

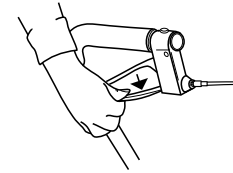- Make sure to engage both brakes when using the parking brakes.

Lock the brakes (parking position):

1. Press the brake handles downwards until a clicking sound is heard.

Unlock the brakes:

1. Pull the brake handles upwards.

4.9 Adjusting the brakes

WARNING!

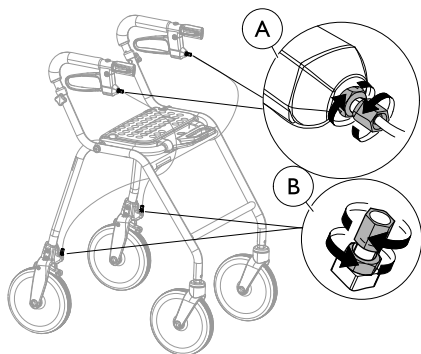
Risk of injury

An improper adjustment of the brakes could lead to a poor braking effect.

- Check the braking effect by engaging the parking brake and ensure that the wheel can't rotate.
- Make sure the locknuts are properly tightened after adjustment.

Adjusting the braking effect

For desirable braking effect, adjust the brake wire.


- The wire can be adjusted in two places - at the top under the brake handle (A) or at the bottom above the brake clamp (B).
- The brake handle must be in neutral position, when adjusting.

8 mm / 0.3 inch and 10 mm / 0.4 inch spanner (not included).

4.10 Flip up seat plate

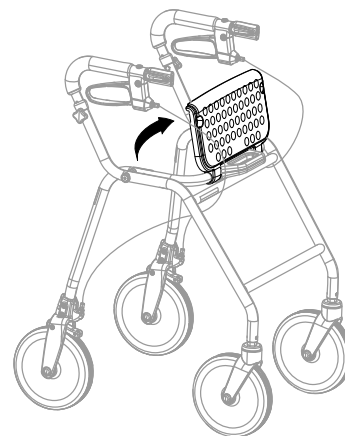Flip up the seat plate

For increased accessibility, the seat can be folded up to provide a larger walking space.

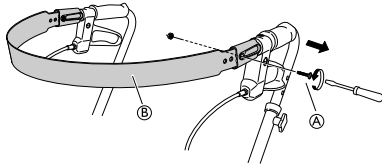
4.11 Accessories

Accessories are not standard on all models. For a list of available accessories, contact your authorized dealer or the Invacare representative (see rear page of this manual).

4.11.1 Padded backrest strap (optional for most versions)


Mounting the padded backrest strap

1.


Loosen and remove the screw and the nut (A)

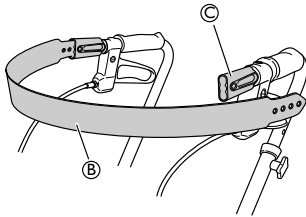2.


Loosen the backrest strap (B) from the attachment (C) on one side.

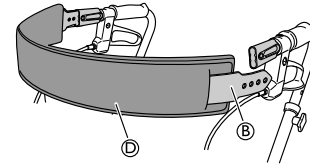3.

Slide the padded backrest strap (D) on the backrest strap (B).

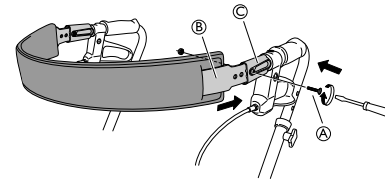4.

Re-mount the backrest strap (B) in the attachment (C) and tighten the screw and nut (A).

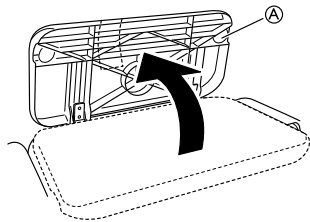
Screwdriver

4.11.2 Padded seat (optional for most versions)


Mounting the padded seat

1.


Fold up the seat plate (A).

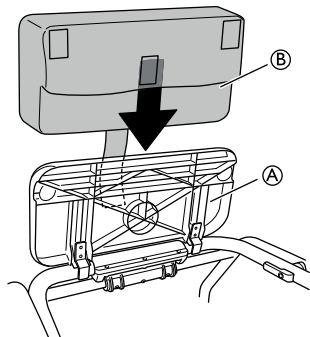2.


Mount the padded seat (B) on the seat plate (A).

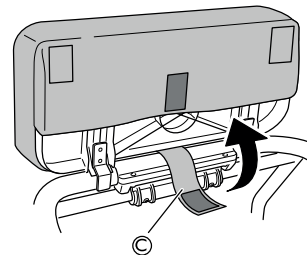3.


Slide the attachment strap (C) between the seat and the folding handle.

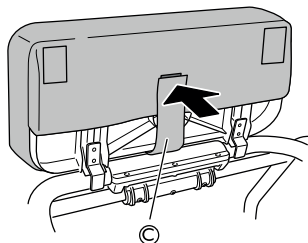4.

Secure the padded seat with the attachment strap ©.

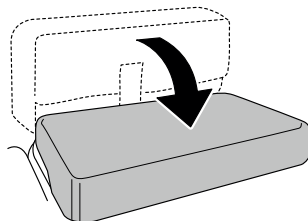5.

Fold down the seat.

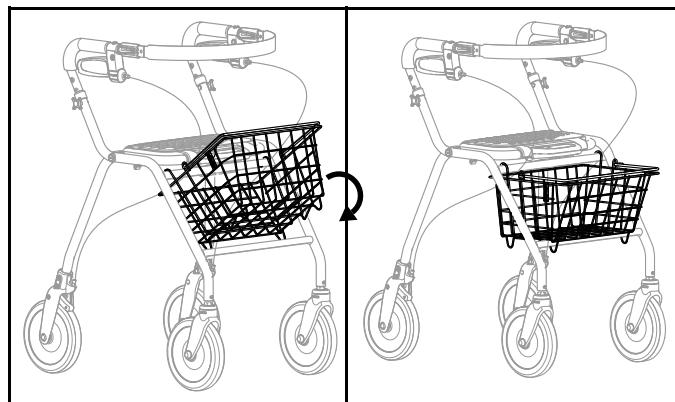
4.11.3 Basket

IMPORTANT!

Do not exceed the maximum allowed load that can be transported in the basket.
– See chapter *Technical data* for maximum load.


Mounting basket

1. Attach the basket to the frame with the hooks.
2. Fold it down and make sure it is properly attached.

4.11.4 Tray

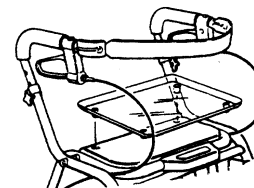Mounting tray

1. Fit the plugs on the downside of the tray into the holes on the seat.
2. Press down the tray.

5 Maintenance

5.1 General Maintenance Information

WARNING!
Risk of Injury

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.
- Ensure that hand grips are tight.
If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation.
If brake adjustment is required, see chapter 4 *Usage*, page 10.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels' tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5.3 Cleaning and Disinfection

5.3.1 General Safety Information

CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.3.2 Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

5.3.3 Cleaning Instructions

The product can be washed in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After washing, wipe the product dry.

Disinfection

- Wipe down all generally accessible surfaces with disinfectant.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

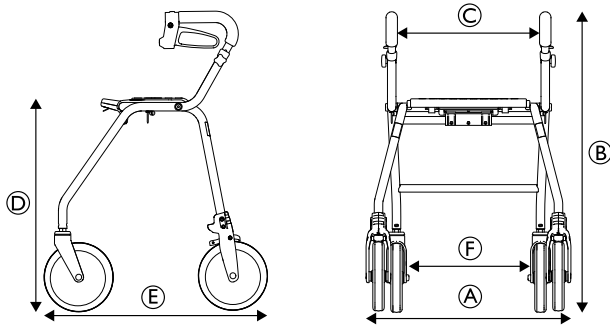
For detailed information, see *5 Maintenance, page 18* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions and Weights

| | Legacy 450 | Legacy 520 | Legacy 600 |
|---------------------------------------|--------------------------|--------------------------|--------------------------|
| Total width Ⓐ | 600 mm / 23.6" | 600 mm / 23.6" | 600 mm / 23.6" |
| Total height Ⓑ | 715–800 mm / 28.1"–31.5" | 760–850 mm / 29.9"–33.5" | 835–920 mm / 32.9"–36.2" |
| Seat width Ⓒ | 420 mm / 16.5" | 420 mm / 16.5" | 420 mm / 16.5" |
| Seat height Ⓓ | 470 mm / 18.5" | 540 mm / 21.3" | 620 mm / 24.4" |
| Total length Ⓔ | 610 mm / 24" | 650 mm / 25.6" | 720 mm / 28.3" |
| Distance between rear wheels Ⓕ | 455 mm / 17.9" | 455 mm / 17.9" | 455 mm / 17.9" |
| Front/ rear wheel diameter | 200 mm / 7.9" | 200 mm / 7.9" | 200 mm / 7.9" |

Weights

| | Legacy 450 | Legacy 520 | Legacy 600 |
|-------------------------------|-------------------|-------------------|-------------------|
| Total weight ¹ | 6.9 kg / 15.2 lbs | 7.2 kg / 15.9 lbs | 7.6 kg / 16 lbs |
| Max. user weight ² | 150 kg / 330 lbs | 150 kg / 330 lbs | 150 kg / 330 lbs |
| Max. load in basket | 7 kg / 15 lbs | 9 kg / 20 lbs | 9 kg / 20 lbs |
| Max. load on tray | 5 kg / 11 lbs | 5 kg / 11 lbs | 5 kg / 11 lbs |

7.2 Material

| | Legacy |
|---------------|--|
| Frame | Polyester (PES) powder coated steel |
| Handle bars | Chrome plated steel |
| Hand grips | Rubber foam |
| Wheels | Polypropylene (PP) with treads of Thermoplastic Elastomers (TPE) |
| Plastic parts | Material according to marking; Polyamide (PA), Polypropylene (PP), Polyethylene (PE) |
| Screws | Steel, rust protected (zinc plated) |

1. Total weight excluding equipment.
2. This weight limitation includes the weight of the user, whether standing or sitting, and any additional load, e.g. the content of the basket.

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare Rea AB