Aquatec® Sorrento

en	Shower Stool / Shower Chair	
	User Manual	2
de	Duschhocker/Duschstuhl	
	Gebrauchsanweisung	5
fr	Tabouret de douche/chaise de douche	
	Manuel d'utilisation	8
it	Sgabello da doccia / Sedia da doccia	
	Manuale d'uso	12
es	Taburete de ducha / Silla de ducha	
	Manual del usuario	15
pt	Banco de duche/Cadeira de duche	
	Manual de utilização	18
nl	Douchekrak / douchestoel	
	Gebruiksaanwijzing	21
da	Brusetaburet/brusestol	
	Brugsanvisning	24
no	Dusjkrakk/dusjstol	
	Bruksanvisning	27
sv	Duschpall/Duschstol	
	Bruksanvisning	30
fi	Suihkujakkara/suihkutuoli	
	Käyttöohje	33

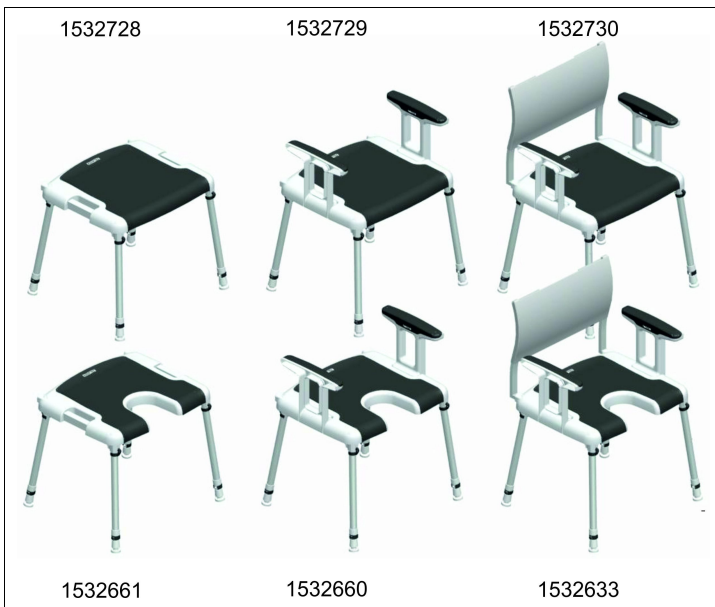
Fig. 1

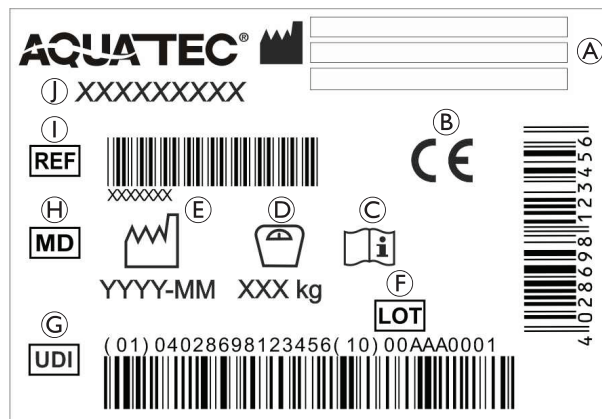
Fig. 2

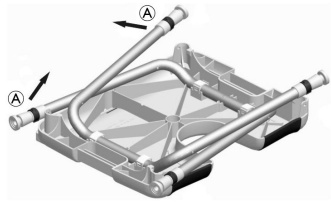
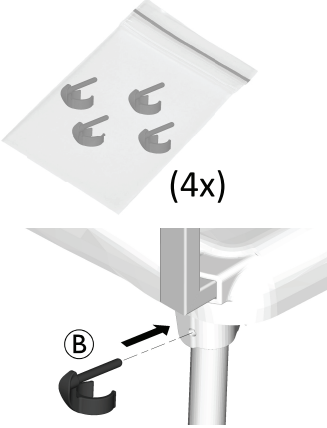
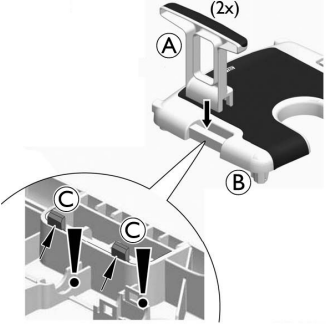
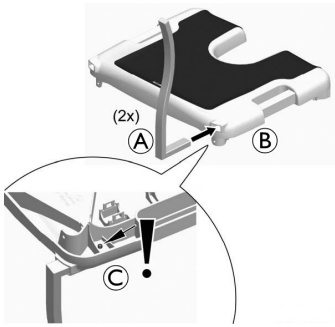
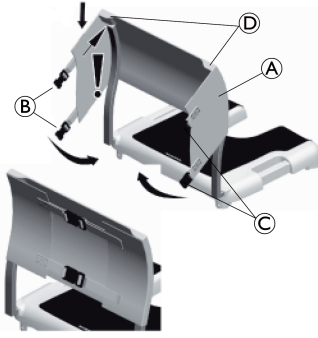
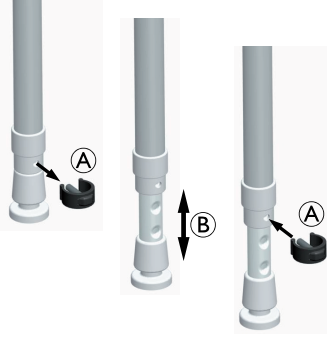
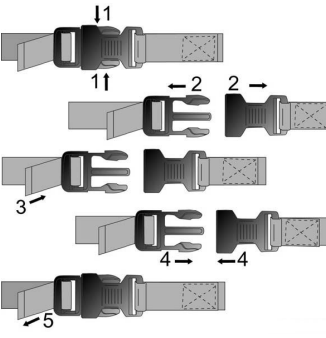
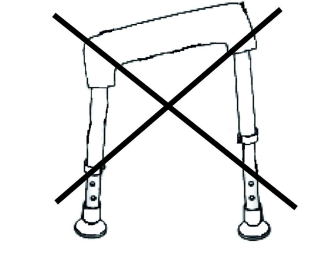
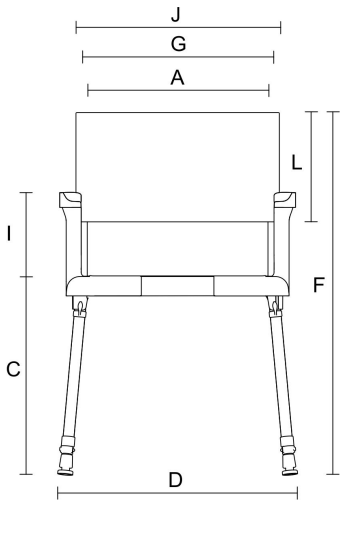
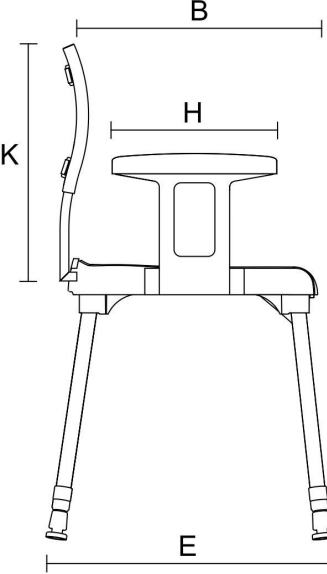
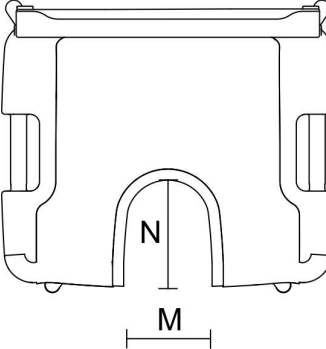
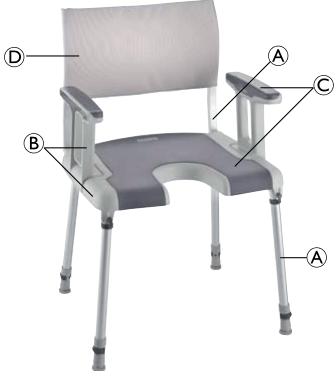
This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

AQUATEC®

Yes, you can.®

			
			
			
<p>Fig. 11</p>	<p>Fig. 12</p>	<p>Fig. 13</p>	<p>Fig. 14</p>

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the

website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not stand on the product.
- Do not use or store the product outdoors.
- Do not tilt.
- Do not sit on the product if not all four legs have full contact to the level ground.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

2.2 Labels and Symbols on the Product (Fig. 2)

Ⓐ	Manufacturer	Ⓕ	Lot number
Ⓑ	European Conformity	Ⓖ	Unique Device Identifier
Ⓒ	Read user manual	Ⓗ	Medical Device
Ⓓ	Maximum user weight	Ⓛ	Reference number
Ⓔ	Production date	Ⓜ	Product designation

The identification label is attached to the underside of the seat plate.

3 Product Overview

3.1 Intended Use

The Aquatec Sorrento is a modular shower chair, intended to be used as a shower aid. It is composed of a stool with four anti-slip height adjustable and foldable legs and a padded plastic seat. The product is available with or without padded detachable armrests, a textile backrest and a hygiene cut-out on the seat.

This product is intended to be used as a shower aid.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indications

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data, page 5*.

3.2 Scope of Delivery (Fig. 1)

- Seat plate with foldable legs
- Armrest (if applicable) (2x)
- Backrest (if applicable)
- Safety clips (4x)
- User manual

The scope of delivery depends on the version ordered. The possible versions are shown in figure 1.

4 Setup (Fig. 3 – 10)

CAUTION!

Risk of Finger Trapping

– Take care during assembly.

1. Fold out legs Ⓐ on both sides, see Fig 3.
2. Secure legs with safety clips Ⓑ on all four sides, see Fig 4.

**CAUTION!****Risk of Injury**

- Ensure that all safety clips are correctly assembled.

Adjusting the Seat Height

The height of the seat can be adjusted between 460 and 610 mm in six steps of 25 mm.

1. Remove safety clips **A** from all four legs, see Fig 8.
2. Adjust the height **B** of the legs, see Fig 8.
3. Re-insert safety clips **A** on all four legs, see Fig 8.

**WARNING!****Risk of Falling**

- Ensure that all legs are adjusted to the same height, see Fig 10.

Fitting the Armrests (Option)

1. Drop armrests **A** into seat plate **B** on both sides, see Fig 5.

**WARNING!****Risk of Injury**

- Ensure that both catches **C** in each armrest audibly engage on the underside of the seat plate, see Fig 5.

**CAUTION!****Risk of Entrapment**

- Do not put your fingers in open holes at the armrest and below the seat.

Fitting the Backrest (Option)

1. Push backrest frame **A** into the rear openings of seat plate **B** on both sides, see Fig 6.

**WARNING!****Risk of Injury**

- Ensure that both catches **C** of the backrest frame audibly engage on the underside of the seat plate, see Fig 6.

2. Pull down backrest cover **A** over both sides of the backrest frame, see Fig 7.

**WARNING!****Risk of Injury**

- Ensure that the ends of the backrest frame are properly located in the recesses **D** on the right and left of the backrest cover, see Fig 7.

3. Close the backrest bands by pushing latches **C** into buckles **B**, see Fig 7.

**WARNING!****Risk of Injury**

- Make sure that all buckles are securely closed.

4. Tighten the backrest bands, see Fig 9.

Tighten or slacken the bands to adjust the firmness of the backrest cover.

The chair is disassembled in reverse order to assembly.

5 Maintenance**5.1 General Maintenance Information**

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection**General Safety Information****CAUTION!****Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**NOTICE!**

- Wrong fluids or methods can harm or damage the product.
 - All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals**NOTICE!**

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination. Clean and disinfect the product:
 - regularly while in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

Cleaning**NOTICE!**

- Maximum cleaning temperature:
 - 60 °C by hand.
 - 85 °C for 3 minutes by automatic washing systems.

1. Remove detachable parts.
2. Clean the product by wiping down all generally accessible surfaces.
3. Dry the parts with a cloth.

Cleaning the Backrest Cover

1. Remove the backrest cover.
2. Machine-wash the cover at max. 60 °C using a mild detergent.

**NOTICE!****Risk of Damage**

- Washing at higher temperatures can cause damage.
- Do not tumble dry.

**NOTICE!**

- Excessive cleaning can damage the textile.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

6.2 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Dimensions, Weights and Materials

Dimensions (Fig. 11, 12, 13)			
(A)	400 mm	(H) ¹	280 mm
(B)	400 mm	(I) ¹	195 mm
(C)	460 - 610 mm	(J) ¹	470 mm
(D)	570 - 600 mm	(K) ¹	380 mm
(E)	490 - 520 mm	(L) ¹	280 mm
(F) ¹	850 - 990 mm	(M) ¹	140 mm
(G) ¹	440 mm	(N) ¹	170 mm

1 if applicable

Weights (Fig. 14)	
Shower stool	3.5 kg
Shower stool with armrest	4 kg
Shower chair	5.5 kg
Max. user weight	135 kg

Materials (Fig. 14)			
(A)	AL	(C)	PP
(B)	PP	(D)	Nylon / PVC coated

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare S.A.
Invacare GmbH