# Invacare® Futura

R8802 / R8802-60 / R8803 / R8803-60 / R8804 / R8804-60

en	Folding Shower Seat User Manual .....	3
de	Duschklappsitz Gebrauchsanweisung .....	5
fr	Siège de douche mural Manuel d'utilisation .....	8
it	Seggiolino da doccia ribaltabile Manuale d'uso .....	11
es	Asiento de ducha plegable Manual del usuario .....	14
pt	Assento do duche rebatível Manual de utilização .....	17
nl	Opklapbaar douchezitje Gebruiksaanwijzing .....	20


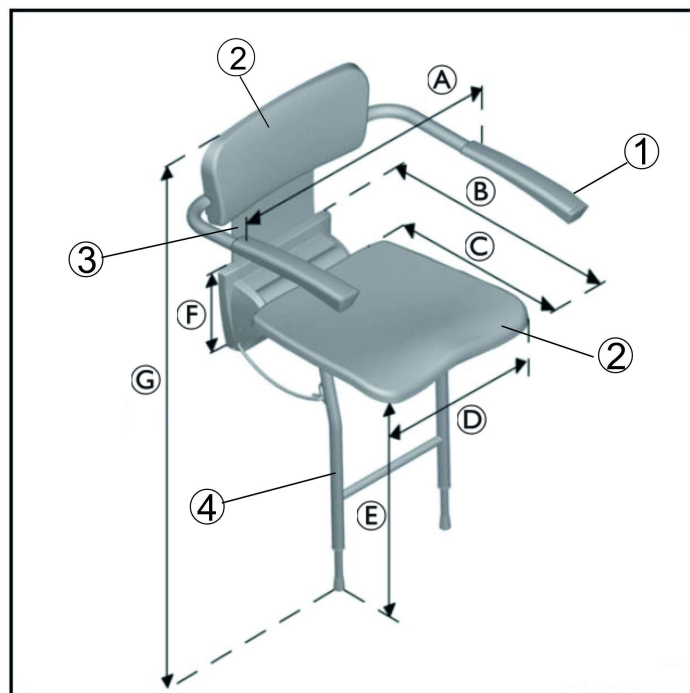
Fig. 1

Fig. 2

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

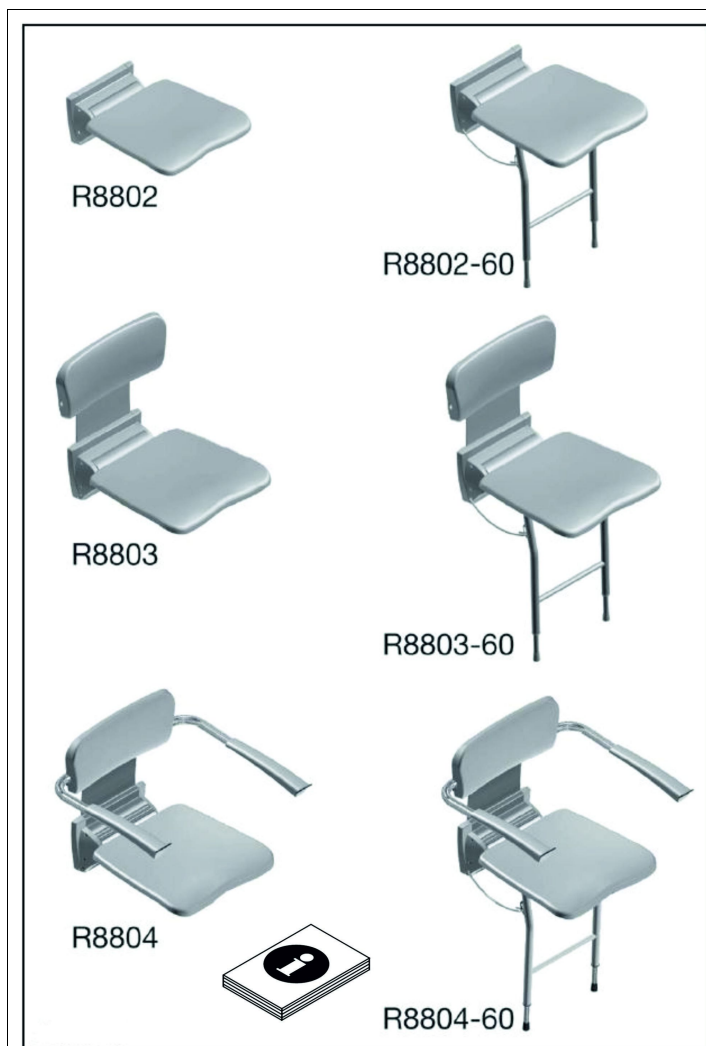Yes, you can.®

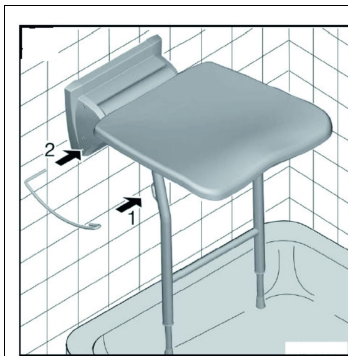
Fig. 18

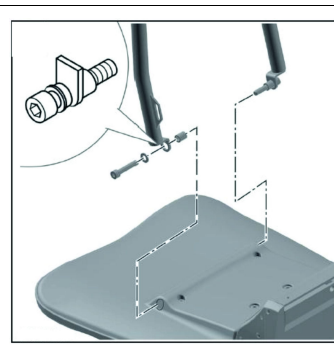
Fig. 19

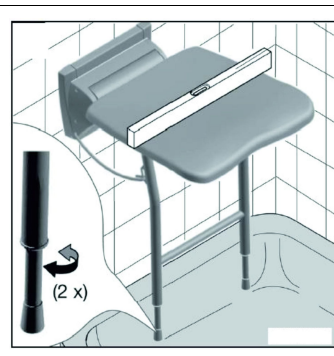
Fig. 20

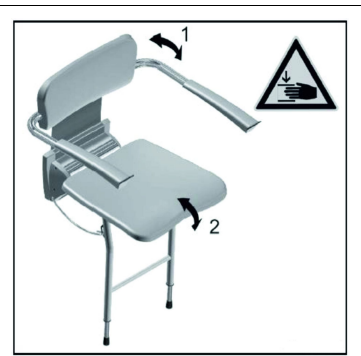
Fig. 21

en

## 1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### 3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1471200-C

## 5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 6 Safety

### 6.1 Safety Information

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not stand on this product.
- Do not use as a climbing aid.
- All legs must be set at the same height. (If applicable)
- Do not use or store the product outdoors.
- Always use new wall plugs and bolts when repositioning the product.

#### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**6.2 Labels and Symbols on the Product (Fig. 3)**

Ⓐ	Manufacturer	Ⓕ	Medical Device
Ⓑ	European Conformity	Ⓖ	Lot number
Ⓒ	Read user manual	Ⓗ	Reference number
Ⓓ	Maximum user weight	Ⓘ	Product designation
Ⓔ	Production date		

The identification label is attached to the wall fixing board.

**7 Product Description**

Invacare Futura series are wall-mounted shower seats available with or without: backrests, support feet (to the ground) and armrests.

**8 Scope of Delivery (Fig. 1)**

The scope of delivery depends on the version ordered. The possible versions are shown in figure 1.

Not all products, models or accessories may be available in every country.

**9 Intended Use**

This product is intended to be used as a shower aid.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips) or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

**Indication**

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 135 kg.

**10 Assembling the Product (Fig. 4 – 21)****Mounting of the Folding Shower Seat (Fig. 4 – 10)**

1. Measure the individual seat height of the user. (Fig 5)

**CAUTION!****Risk of Injury**

- Mount only to walls that have a sufficient load-bearing capacity. (Fig. 4)
- Use only suitable fastening materials.

2. Open the headless screws with an Allen key then remove the wall mounting plate. (Fig. 6)
3. Measure the depth of the shower tray. (Fig. 7)

4. Drill two holes at the calculated height using the wall fix board as a pattern. Also drill the third hole on the bottom (10 mm). (Fig 8)
5. Fix the board to the wall. Use screws, washers and plugs, suitable for the consisting underground. (Fig. 9)
6. Fix the shower seat to the board as shown in the image. Tighten the headless screws on the underside by using an Allen key. Attach the capping. (Fig. 10)

**Mounting the Backrest (Optional) (Fig. 11 – 16)**

1. Remove the backrest from the backrest holder. (Fig. 11)
2. Put the spacer plate into the designated notch in the wall mounting plate. (Fig. 12)
3. Cover the seatplate with a cloth. Drill three holes (10 mm) using the back rest holder as a pattern. (Fig. 13)
4. Fix the board to the wall. Use screws, washers and plugs, suitable for the consisting underground. (Fig. 14)
5. Snap on the cappings. (Fig. 15)
6. Snap on the backrest. (Fig. 16)

If applicable: The backrest with the preassembled armrest is mounted in the same way as the backrest.

The device is dismantled by following the assembly procedure in reverse order.

**Mounting the Legs (Upgrade-Kit) (Fig. 17 – 19)**

1. Drill two holes (6 mm) at the marked spot. (Fig. 17)
2. Break through the designated holes on the underside of the seatplate. (Fig. 17)
3. Fix the bracket as shown in the image. (Fig. 18)
4. Attach the support feet by using the included screws and nuts. (Fig. 19)

**Adjusting the Legs (Fig. 20)**

1. Rotate the leg extension tip counterclockwise (for lengthening) or clockwise (for shortening). Use a spirit level to level out correctly. (Fig. 20)

**11 Usage (Fig. 21)****CAUTION!****Risk of Trapping/Squeezing**

Do not reach into the folding mechanism. (Fig. 21)

**Folding the Shower Seat**

1. Fold the shower seat by grabbing the armrests and moving them upward. (Fig. 21) (If applicable)
2. Fold up the seatplate.

**Unfolding the Shower Seat**

1. Unfold the shower seat by grabbing the armrests and moving them downward. (Fig. 21) (If applicable)
2. Fold down the seatplate.

**12 Maintenance****12.1 General Maintenance Information**

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

**12.2 Cleaning and Disinfection****General Safety Information****CAUTION!****Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

- ! IMPORTANT!**  
Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
  - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
  - Always make sure that the product is completely dried before taking it into use again.

### Cleaning Intervals

- ! IMPORTANT!**  
Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination. Clean and disinfect the product
- regularly while it is in use,
  - before and after any service procedure,
  - when it has been in contact with any body fluids,
  - before using it for a new user.

### Cleaning Instructions

- ! IMPORTANT!**
- Clean by hand
  - Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

### Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

### 13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *12 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

### 14 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling

in each country. Contact your local waste management company for information.

## 15 Technical Data (Fig. 2)

Dimensions, unfolded			
Ⓐ	385 – 545 mm	Ⓔ	440 – 550 mm
Ⓑ	470 mm	Ⓕ	180 mm
Ⓒ	370 mm	Ⓖ	770 – 880 mm
Ⓓ	380 mm		

Dimensions, folded			
Depth when folded	105 – 115 mm	Height when folded	520 – 1210 mm

Weights	
Futura Series	4.8 – 9.1 kg
Maximum user weight Futura	135 kg

Materials			
①	PVC	③	Aluminium
②	ABS	④	Steel, powder coated

All product components are corrosion resistant and do not contain natural rubber latex.

## 16 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Limited