
# Aquatec® PICO GREEN

PICO GREEN / PICO 3in1 GREEN

<b>en</b>	<b>Shower Chair / Shower and Toilet Chair</b> User Manual .....	<b>3</b>
<b>de</b>	<b>Duschstuhl / Dusch- und Toilettenstuhl</b> Gebrauchsanweisung .....	<b>5</b>
<b>fr</b>	<b>Chaise de douche/Chaise de douche et de toilettes</b> Manuel d'utilisation .....	<b>9</b>
<b>it</b>	<b>Sedia da doccia / Sedia da doccia e da toilette</b> Manuale d'uso .....	<b>12</b>
<b>nl</b>	<b>Douchestoel / douche- en toiletstoel</b> Gebruiksaanwijzing .....	<b>15</b>
<b>da</b>	<b>Brusestol/bruse- og toiletstol</b> Brugsanvisning .....	<b>18</b>
<b>fi</b>	<b>Suihkutuoli / suihku- ja WC-tuoli</b> Käyttöohje .....	<b>21</b>
<b>sv</b>	<b>Duschstol/dusch- och toalettstol</b> Bruksanvisning .....	<b>24</b>
<b>no</b>	<b>Dusjstol / Dusj- og toalettstol</b> Bruksanvisning .....	<b>27</b>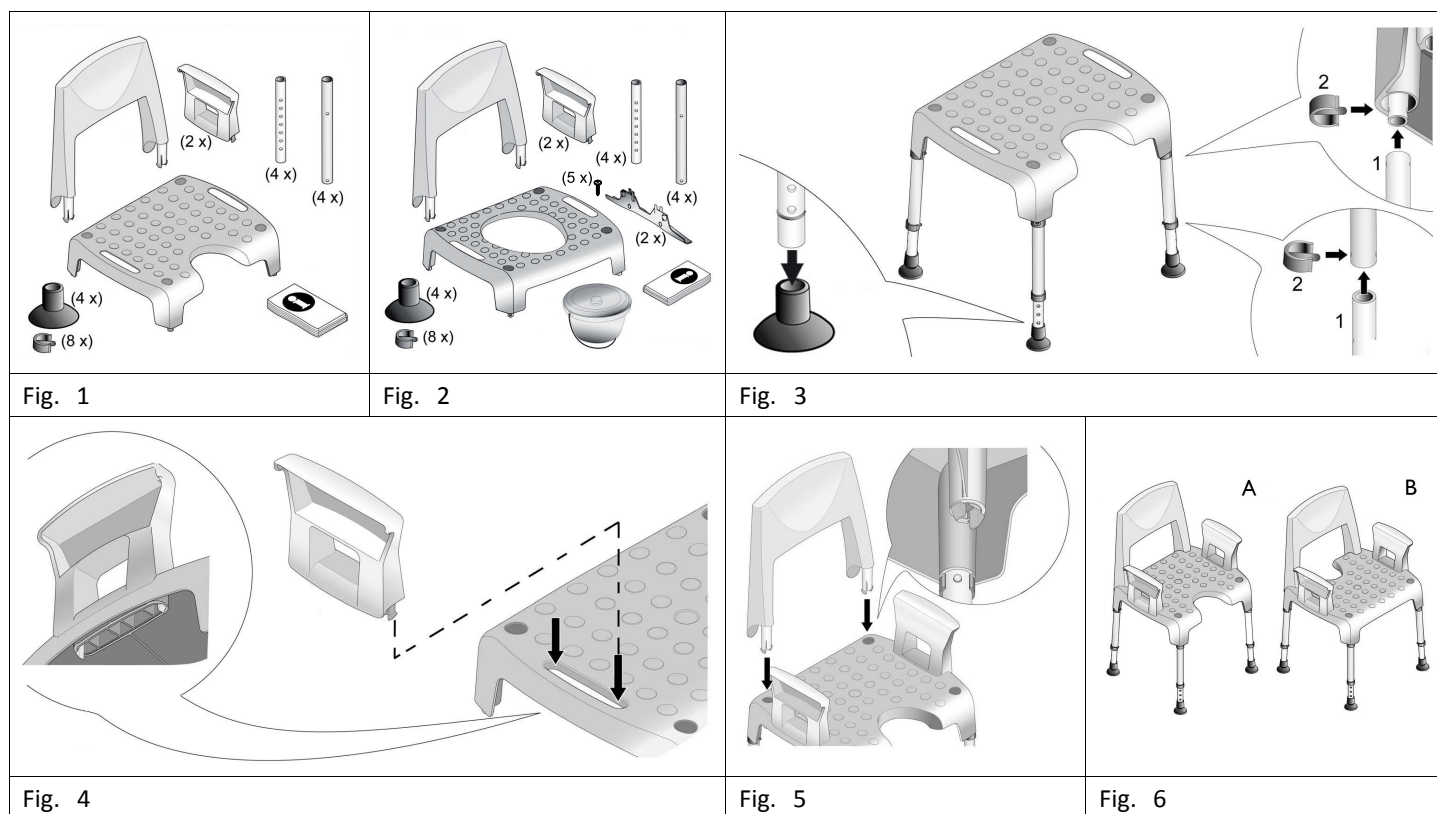

This manual **MUST** be given to the user of the product.  
BEFORE using this product, this manual **MUST** be read and saved for future reference.

**AQUATEC®**

**Yes, you can.®**

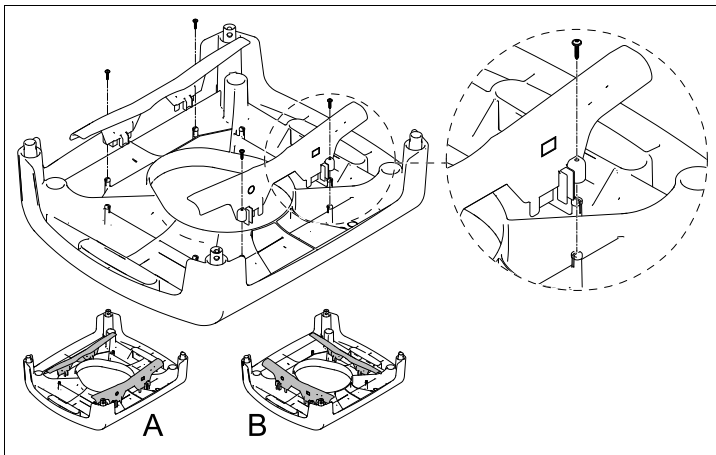
Fig. 7

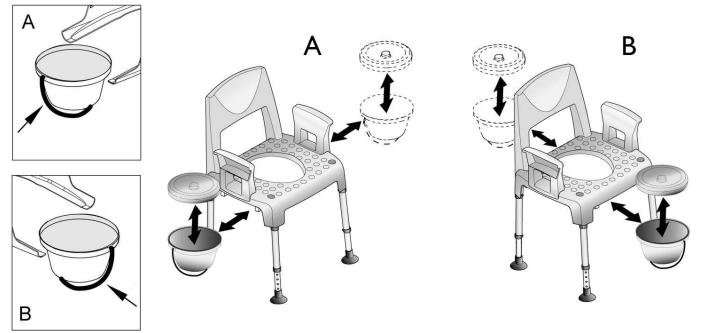
Fig. 8

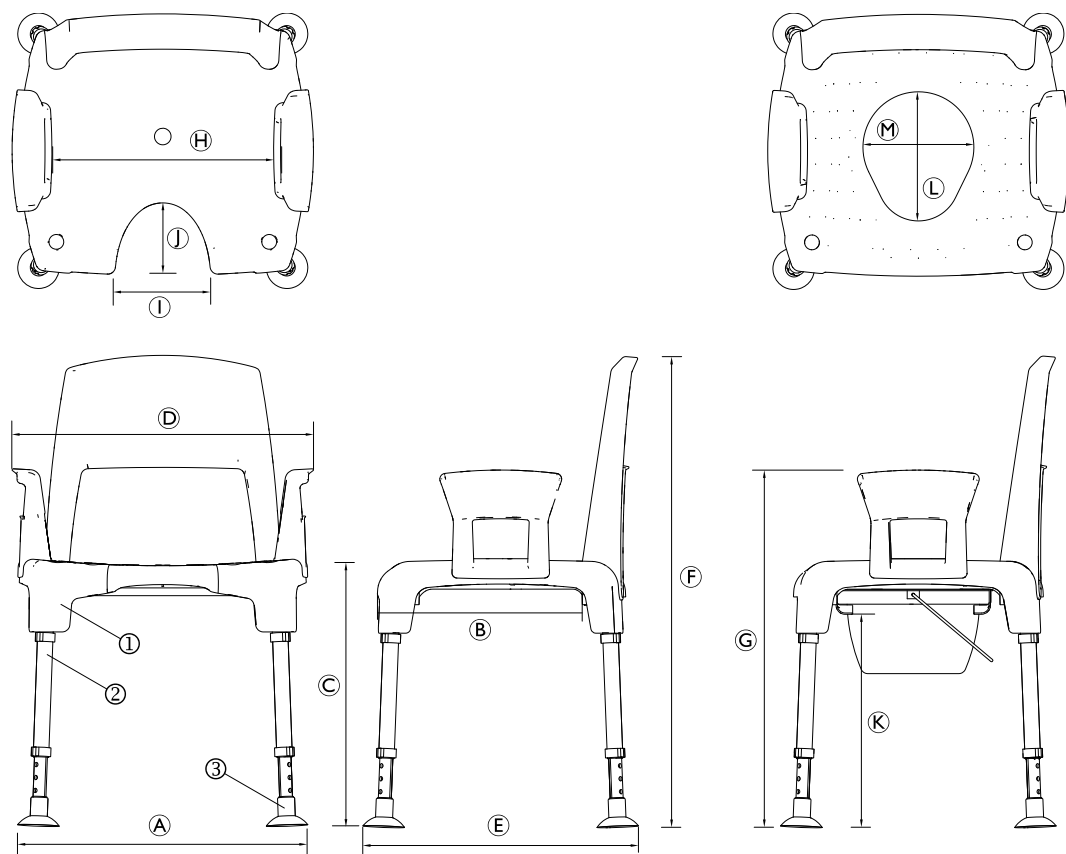
Fig. 9

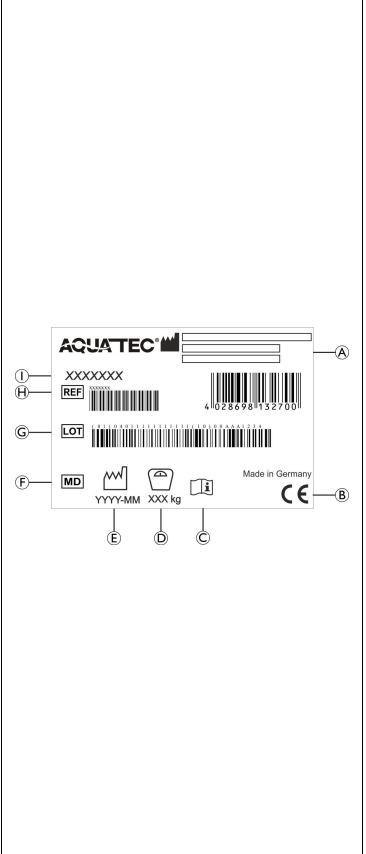
Fig. 10

## 1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We only use REACH compliant materials and components.

### 3.1 Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 6 Safety

### 6.1 Safety Information

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.
- Only use accessories intended for this product.
- Do not stand on this product.
- Do not use as a climbing aid.

#### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

#### CAUTION!

##### Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.

### 6.2 Labels and Symbols on the Product (Fig. 10)

(A)	Manufacturer	(F)	Medical Device
(B)	European Conformity	(G)	Lot number
(C)	Read user manual	(H)	Reference number
(D)	Maximum user weight	(I)	Product designation
(E)	Production date		

The identification label is attached to the underside of the product.

## 7 Scope of Delivery

PICO GREEN (Fig. 1):

- Seat plate
- Backrest
- Armrest (2x)
- Inner leg tubing (4x)
- Outer leg tubing (4x)
- Clip (8x)
- Anti-slip foot (4x)
- User manual

PICO 3in1 GREEN (Fig. 2):

- Seat plate
- Backrest
- Armrest (2x)
- Inner leg tubing (4x)
- Outer leg tubing (4x)
- Clip (8x)
- Anti-slip foot (4x)
- User manual
- Toilet pan with lid
- Toilet pan holder (2x)
- Screws (5x)

## 8 Intended Use

Aquatec® PICO GREEN is intended to be used as a shower aid.

Users are any adults and adolescents with restricted or limited movements (e.g.: legs and/or hips). If used without an assistant, the users must be able to keep their own balance.

### Indication

Restricted or limited movements (e.g.: legs and/or hips) or having balance issue to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 160 kg.

## 9 Product Description

Aquatec® PICO GREEN is a modular plastic shower chair. It is composed of a stool with hygiene recess, with four anti-slip and height-adjustable feet, two removable armrests and one removable backrest.

Aquatec® PICO 3in1 GREEN is a modular plastic shower- and toilet chair. It is composed of a stool with toilet opening, with four anti-slip and height-adjustable feet, two removable armrests and one removable backrest.

## 10 Assembling the Product

### Installing the Feet (Fig. 3)

#### CAUTION!

##### Risk of Trapping

- Take care during assembly.

1. Push the four anti-slip feet completely onto the height-adjustable telescope tubes.
2. Install the four height-adjustable telescope tubes to the legs and secure with clips.
3. Install the four assembled legs to the tube ends on the underside of the seat and secure with clips.


#### CAUTION!

##### Risk of Injury or Damage

- Make sure that all eight clips are securely in place.

All four legs generally should be adjusted to the same height. As an exception, the front legs may be lowered by one position to improve the seating comfort when restricted posture is given due to problems with the coccyx etc.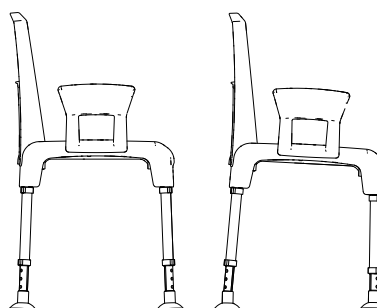

#### WARNING!

##### Risk of Tilting

- Never adjust the rear legs to a lower position than the front legs.
- When lowering the front legs, make sure that they are lowered by maximum one position.

### Installing the Armrests and/or the Backrest (Fig. 4, 5)

1. Insert the armrests from above into the seat plate until they engage with an audible click (Fig. 4).
2. Insert the backrest from above into the seat plate until it engages with an audible click (Fig. 5). The backrest can be installed with the hygiene recess facing forward or backward. (Fig. 6)

To remove the armrests/backrest press together the snap-fits on the underside of the seat and pull out the armrests/backrest.

### Installing the toilet pan holders (Fig. 7, 8)

The toilet pan holders can be mounted crosswise (A) or lengthwise (B).

Five screws are included in the scope of delivery (Fig. 2) for assembling the two toilet pan holders. Only four screws are required for assembly.

1. Install the toilet pan holders using the delivered screws.

#### IMPORTANT!

##### Risk of damage to the product

- The plastic parts can crack when the screws are overtightened.
- Tighten the screws by hand only.

2. Slide the toilet pan into the holders under the seat.

The product is disassembled in reverse order to assembly.

## 11 Maintenance

### 11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

### 11.2 Cleaning and Disinfection

#### General Safety Information

#### CAUTION!

##### Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

**IMPORTANT!**

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
  - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
  - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

**Cleaning Intervals****IMPORTANT!**

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.  
Clean and disinfect the product
- regularly while it is in use,
  - before and after any service procedure,
  - when it has been in contact with any body fluids,
  - before using it for a new user.

**Cleaning Instructions****IMPORTANT!**

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

**Disinfection Instructions**

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

**In Domestic Care**

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

**In Clinical and Long Term Care**

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

**12 Reconditioning**

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

**13 Disposal**

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

**14 Technical Data (Fig. 9)**

Dimensions			
Ⓐ	550 – 560 mm	Ⓗ	430 mm
Ⓑ	445 mm	Ⓛ	180 mm
Ⓒ	430 – 580 mm	Ⓜ	145 mm
Ⓓ	580 mm	Ⓚ	325 – 475 mm
Ⓔ	535 – 545 mm	Ⓛ	250 mm
Ⓕ	825 – 975 mm	Ⓜ	215 mm
Ⓖ	605 – 755 mm		

Weights	
Shower chair	4.6 kg
Shower and toilet chair with toilet pan	5.4 kg
Maximum user weight	160 kg

Materials			
①	PP with wood fibre	③	TPE
②	Aluminum, corrosion-free		

All product components are corrosion resistant and do not contain natural rubber latex.

**15 Environmental Parameters**

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Austria GmbH
Invacare AG
Invacare New Zealand Ltd