Invacare® Omega Eco

H440

en	Stationary Toilet Chair User Manual	3
de	Stationärer Toilettenstuhl Gebrauchsanweisung	5
fr	Chaise-toilettes posée fixe Manuel d'utilisation	8
it	Sedia da toilette fissa Manuale d'uso	10
es	Silla inodora Manual del usuario	13
pt	Cadeira sanitária estacionária Manual de utilização	16
nl	Vaste toiletstoel Gebruiksaanwijzing	19
cs	Pevná židle na toaletu Návod k obsluze	21

<p>Fig. 1</p>	<p>Fig. 2</p>
<p>Fig. 3</p>	<p>Fig. 4</p>

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

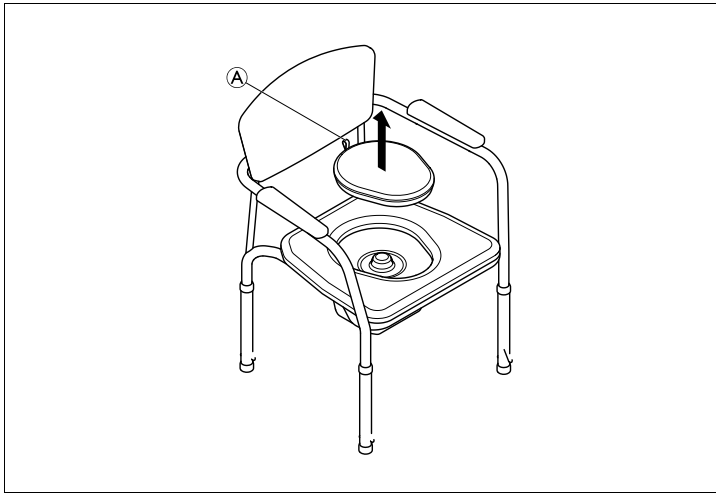
Fig. 5

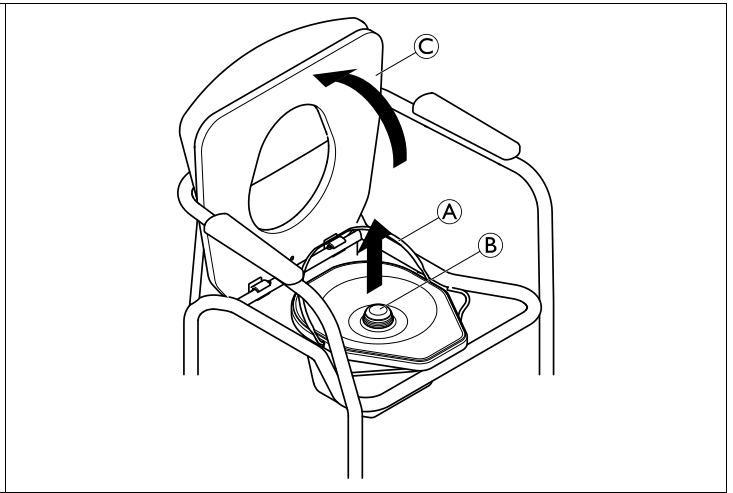
Fig. 6

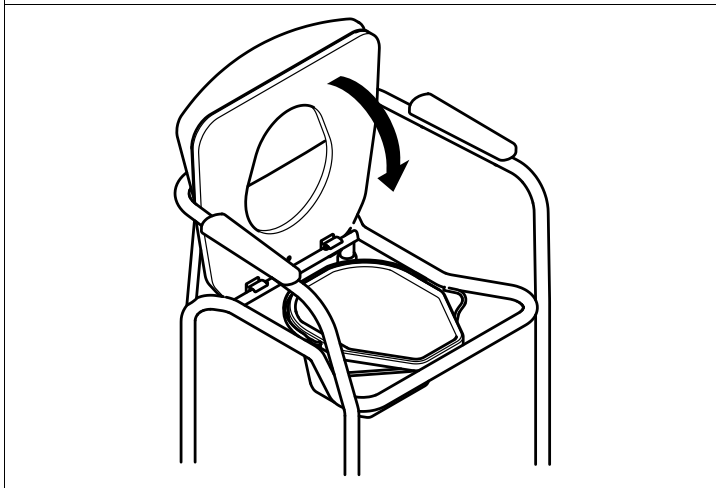
Fig. 7

Fig. 8

Fig. 9

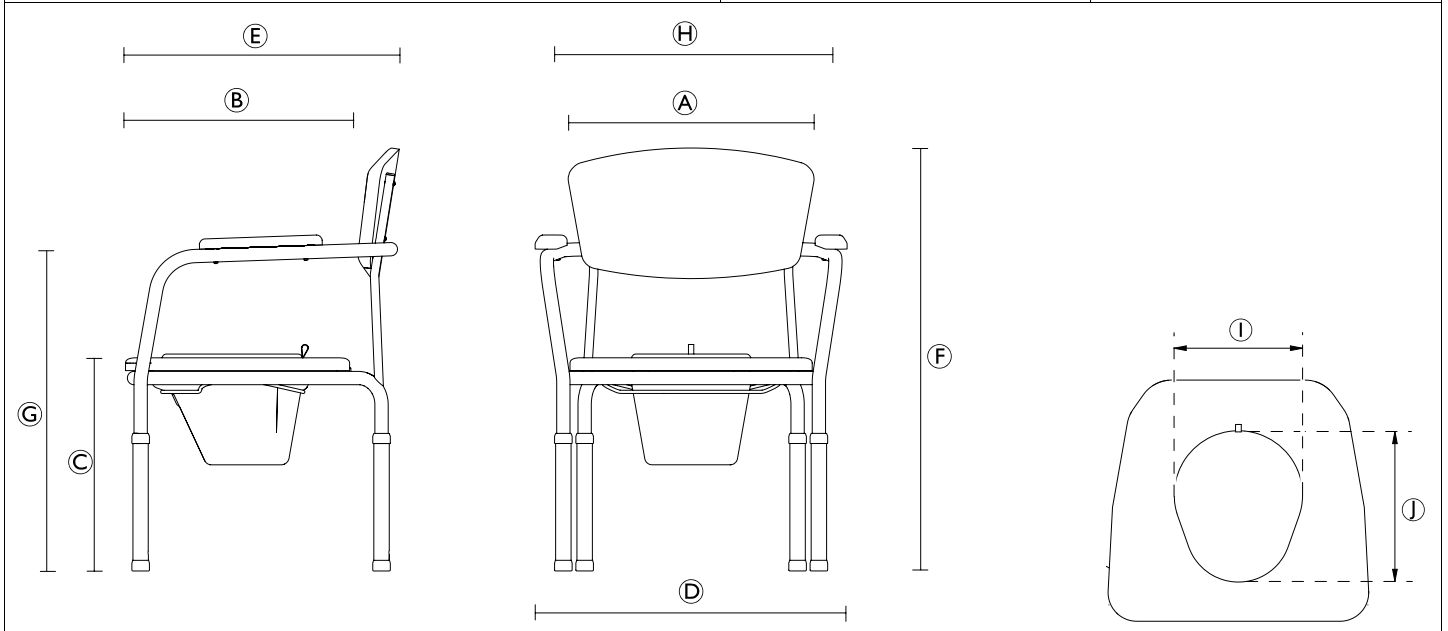
Fig. 10

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Injury!

Improper use of this product may cause injury.

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use or store the product outdoors.
- Do not use in the shower (Fig. 9).
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.

WARNING!

Risk of burning your Skin!

The product components can heat up when exposed to external sources of heat.

- Do not expose the product to strong sunlight before use.

WARNING!

Risk of Falling!

Improper use of this product may cause injury.

- Do not stand on this product.
- Do not move or displace the product under load or during use.
- Do not sit on the product if not all four legs have full contact to the level ground.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor.

2.2 Labels and Symbols on the Product (Fig. 2)

(A)	Manufacturer	(G)	Lot number
(B)	European Conformity	(H)	Unique Device Identifier
(C)	For use on level ground only	(I)	Medical Device
(D)	Read user manual	(J)	Reference number

Ⓔ	Maximum user weight	Ⓚ	Product designation
Ⓕ	Production date		

 The identification label is located under the seat on the tube.

3 Product Overview

3.1 Intended Use

The Invacare Omega Eco H440 is a height adjustable toilet chair composed of padded armrests, backrest, seat plate with toilet opening and seat insert. It can be used over the toilet or independently with a pan with lid. This product is intended to be used as a toilet aid providing stable seating position while going to toilet.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data, page 5*.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

CAUTION!

Risk of Pinching

- Take care during assembly.

Assemble Toilet Seat (Fig. 3)

1. Install the toilet seat plate with the clips Ⓐ to the rear frame tube with an audible click. Make sure the clips engage next to the screws Ⓑ.
2. Position the seat insert into the cutout of the toilet seat plate. By pulling the strap Ⓐ the seat insert can be removed (Fig. 5).

Adjusting Height of Legs (Fig. 4)

1. Press pin Ⓐ and move all adjustment tubes Ⓑ to desired height.
2. Let pin engage at desired height with an audible click.

WARNING!

Risk of Injury

- Make sure that all legs are adjusted to the same height (Fig. 8).
- Ensure that the pins are correctly engaged after adjustment.

Inserting/Removing Toilet Pan (Fig. 6)

The toilet pan can be inserted/removed with or without the lid Ⓑ.

1. Fold up the toilet seat plate Ⓒ (Fig. 6).
2. Use toilet pan handle Ⓐ to insert/remove toilet pan.
3. Fold down toilet seat (Fig. 7).

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- before first use,
- regularly while it is in use,
- when it has been in contact with any body fluids.

Cleaning Instructions

NOTICE!

Maximum cleaning temperature:

- 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions, Weights and Materials (Fig. 10)

Dimensions			
Ⓐ	450 mm	Ⓕ	790 – 970 mm
Ⓑ	425 mm	Ⓖ	620 – 800 mm
Ⓒ	405 – 585 mm (in 20 mm steps)	Ⓗ	470 mm
Ⓓ	585 mm	Ⓘ	220 mm
Ⓔ	515 mm	Ⓚ	250 mm

Weights	
Product weight	8 kg
Max. user weight	140 kg

Materials	
Frame	Steel, powder coated
Upholstery	PVC
Toilet pan	PP

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Poirier SAS
Invacare AG
Invacare GmbH