Aquatec® Pure Bidet

en	Shower toilet	
	User Manual	2
de	WC-Aufsatz mit Wascheinrichtung	
	Gebrauchsanweisung	12
fr	Bidet	
	Manuel d'utilisation	24
it	Bidet multifunzione per WC	
	Manuale d'uso	36
es	Bidé con aspersor	
	Manual del usuario	47
pt	Tampo sanitário	
	Manual de utilização	58
nl	Douchetoilet	
	Gebruiksaanwijzing	69
sv	Bidésits	
	Bruksanvisning	80
da	Bidet	
	Bruksanvisning	91

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

Previous product versions may not be described in this Manual's current revision. If you require assistance, please contact Invacare.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in This Manual

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

DANGER!

Indicates a hazardous situation that will result in serious injury or death if it is not avoided.

WARNING!

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION!

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE!

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations, and information for efficient, trouble-free use.

Other Symbols

(Not applicable for all manuals)

Triman

Indicates recycling and sorting rules (only relevant for France).

1.3 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.4 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.4.1 Product-specific Standards

The product meets the requirements of EN 1717 (Protection against pollution of potable water installations and general requirements of devices to prevent pollution by backflow).

For further information about local standards and regulations, contact your local Invacare distributor.

1.5 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by purchaser or third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional

equipment without first completely reading and understanding these instructions and any additional instructional material.

**WARNING!****Risk of injury**

- Never block the air openings of the product and keep the air openings free of lint, hair, and the like.
- Do not cover up the warm air outlet or insert hands or other objects.
- Never use while sleeping or drowsy.
- Do not operate where aerosol(spray) products are being used or where oxygen is being administered.
- Keep the unit away from fires or inflammable substances.
- Do not put any lighted cigarette or burning material in the toilet.
- Make sure that children do not play with the product.
- Do not connect product to industrial water supplies.
- Never stand on or place objects on the product.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor.

**NOTICE!****Risk of damage to the product**

- Do not press down the lid.
- Do not lean on the lid.
- Do not sit on the lid.
- When not in use for a long period of time, disconnect power plug and drain water tank.
- Do not twist or pull the connection hoses.

2.2 Safety Information for Electrical System

**DANGER!****Electric shock hazard**

- Do not open or repair the product. In case of damages, consult your provider.
- Damaged power cords must be replaced by the manufacturer or your provider.
- Never operate this product if it has a damaged or loosed power cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water
- Do not make any unauthorized changes or alterations to the product.
- Do not use the product while bathing and do not pour or spray water on the unit.
- Always unplug the appliance during cleaning.
- Do not touch the power plug with wet hands.
- Do not reach for a product that has fallen into water. Unplug immediately.
- Do not place or store product where it can fall or be pulled into a tub or sink and do not place or drop product into water or other liquid.
- Never drop or insert any object into any opening or hose.

- Keep the power cord away from heated surfaces.
- Only connect the product to an outlet corresponding to the rated voltage of the unit.
- Before removing the product from the toilet bowl, disconnect the power plug first.

2.3 Safety Information on Electromagnetic Compatibility

This product was successfully tested to international standards for its electromagnetic compatibility. However, electromagnetic fields such as those generated by radio and television transmitters, radio equipment and mobile telephones, can influence the operation of products with electrical drives. The electronic used in this product can also cause weak electromagnetic interference but this lies below the statutory limits. You should therefore observe the following warnings:

**CAUTION!****Electromagnetic interference**

Wireless communications equipment such as WLAN routers, mobile phones, cordless telephones and their base stations or walkie-talkies can affect this equipment.

- Make sure that such devices are kept in a distance of at least 2 m away from the product.
- If other electrically operated medical products cause an interference or are themselves disrupted, switch off your Invacare product and contact your provider.

Guidance and manufacturer's declaration relating to electromagnetic emission and immunity are available on request from Invacare in your country.

3 Product Overview

3.1 Intended Use

Aquatec Pure Bidet is an electrical shower toilet intended to be used as an aid for self-hygiene care and toilet purposes. It is composed of a water spray, an air spray, a heatable toilet seat, a slow down lid brake and a remote control.

Intended Users

Users are any persons with restricted or limited mobility. The users must be able to keep their own balance or ensure the availability of additional support, i.e. wall mounted grab bar. The product may be used without an assistant only by persons who are physically and mentally able to safely deal with it.


Indications

- Limited functions in the arms and hands or
- not having arms and hands or
- having special cleaning demands (e.g.: by water).


There are no contraindications known when using the product as intended. For the maximum user weight, see *9.1 General Data, 11*.

3.2 Main Parts of the Product

The following components are included within the scope of delivery: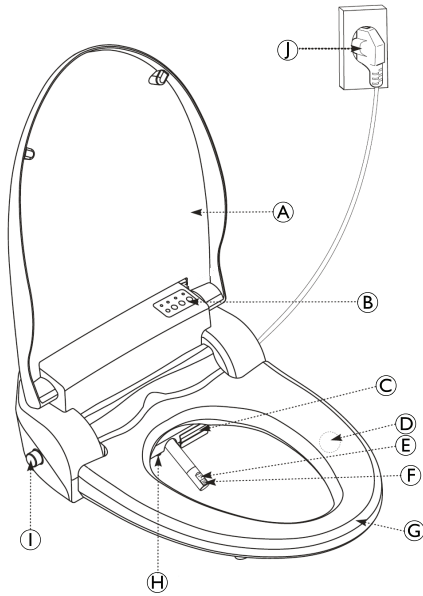


A	Lid	F	Bidet nozzle
B	Control panel	G	Heated seat
C	Outlet of warm air dryer	H	LED Night-light
D	Seat sensor	I	Inlet adapter
E	Cleansing nozzle	J	Power plug

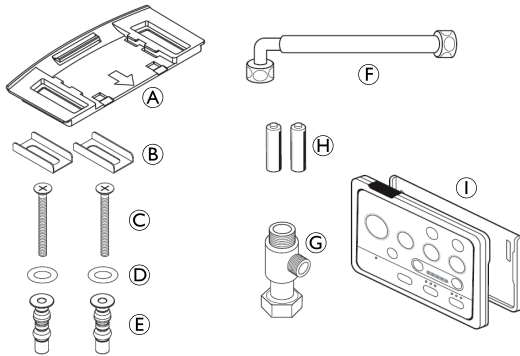A	Catch plate	F	PVC Flexible hose (for main unit)
B	Brackets (2x)	G	T-valve
C	Fixation bolts (2x)	H	Batteries for remote control
D	Washers (2x)	I	Remote control and holder
E	Packing nuts (2x)		

3.3 Labels and Symbols on the Product

The identification label is applied to the rear side of the product and contains the following information:

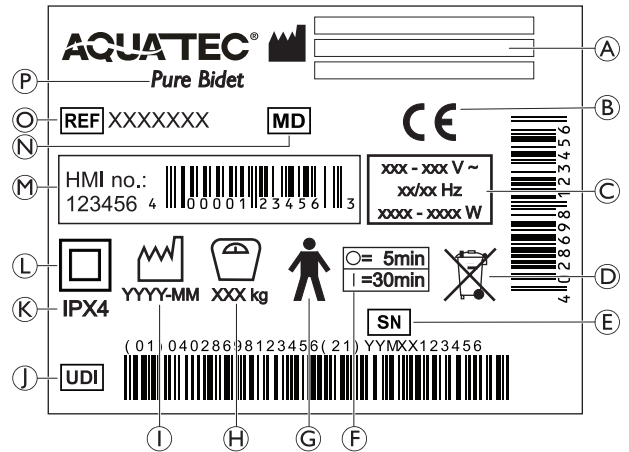A	Manufacturer	I	Date of manufacture
B	European Conformity	J	Unique Device Identifier
C	Nominal voltage / rated power	K	Protection class
D	WEEE conform	L	Class II equipment
E	Serial number	M	HMI number
F	Interval of use: Max. 30 minutes continuous use followed by a 5 minutes break.	N	Medical device
G	Type B applied part	O	Reference number
H	Maximum user weight	P	Device designation

	Read user manual (white symbol on blue background)
--	--

4 Setup

4.1 Installation / Assembly

WARNING!
Electric Shock Hazard

— Do not connect power until installation is finalized.

WARNING!
Risk of being scalded by hot water


— Only connect shower toilet to cold water supply.

The Aquatec Pure Bidet fits standard toilets. It is suitable for all types of water supply connection with a flushing cistern (top, bottom, sideways).

In some regions, tap water contains particularly high levels of lime. Limescale build-up can impair the function of the product and promote the spread of germs. Systems for softening the tap water or the installation of a limescale filter prevents limescale build-up deposits and can have a positive effect on the lifetime of the product.


Preparing the Toilet

1.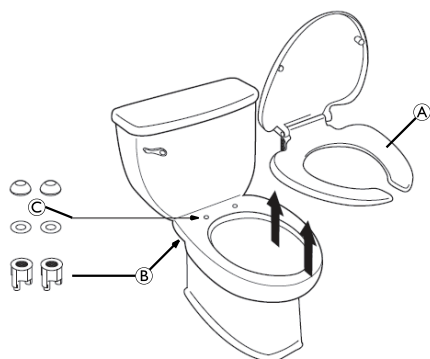


Remove the existing toilet seat **A** and all its fixation elements **B** and **C** from the toilet bowl.

2.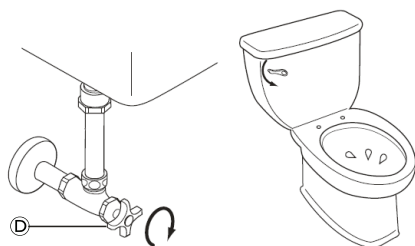

Close the shut off valve **D** clockwise and drain the toilet water tank completely by flushing.

3.

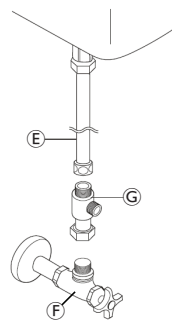Detach the water supply hose **E** on the part of the shut off valve **F**.

If part **E** is a rigid pipe, it must be replaced by a flexible hose.


4. Mount the T-valve **G** between shut off valve and water supply hose.

If the T-valve does not fit onto the water supply, adapter pieces are necessary.


Assembling the Shower Toilet

1.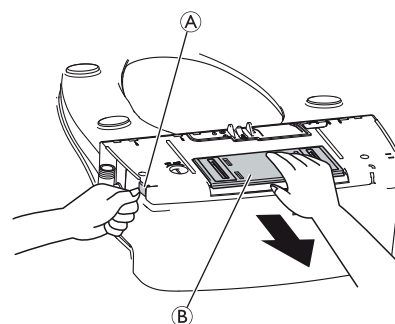

Press lever **A** to detach the catch plate **B** from the bidet.

2.

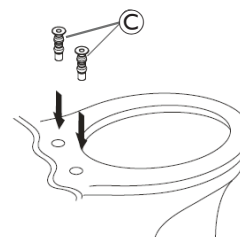
Insert the supplied packing nuts **C** into the fixing holes on the toilet bowl.

One pair of standard packing nuts with a diameter of 14 mm are included. If the fixing hole diameter on the toilet bowl is less than 14 mm other suitable packing nuts and washers need to be used.


3.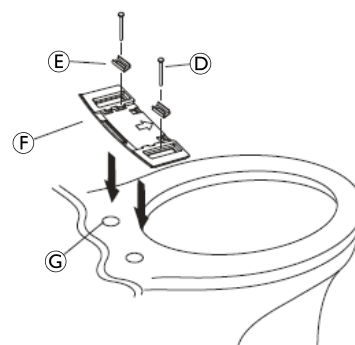

Put the washers **G** on both packing nuts.

4. Set in place the brackets **E** on the catch plate **F**.

- Fasten the catch plate with the fixation bolts ① through the brackets in both packing nuts.
-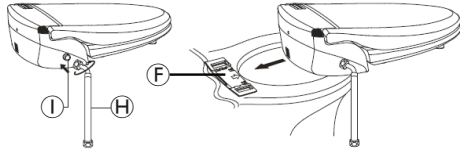

Screw the PVC flexible hose ⑧ clockwise on the inlet adapter ① of the shower toilet.


- Slide the shower toilet to the catch plate ⑥ installed.

CAUTION!

— Make sure that the grooves underneath the shower toilet are precisely guided in the recesses of the catch plate.


-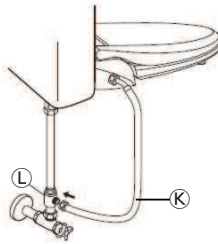

Screw the PVC flexible hose ⑧ clockwise on the T-valve ①.


Assembling the Remote Control

-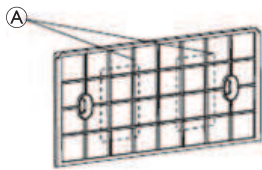

Attach the remote control holder with double stick tape ① on the wall in a suitable location.

-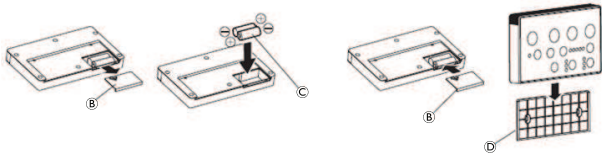

Remove battery cover ② from remote control.

- Insert batteries ③. Ensure correct polarity.
- Close battery cover of remote control.
- Place remote control into its holder ④.

4.2 Assembling Concealed Cistern (optional)

Installing Water Flow over Angle Valve


- Connect angle valve and bidet with the PVC Flexible hose.
- Turn the water tap on.

NOTICE!

— Check the connection for leaks and re-fit if necessary.

Installing Water Flow over Flush-mounted Housing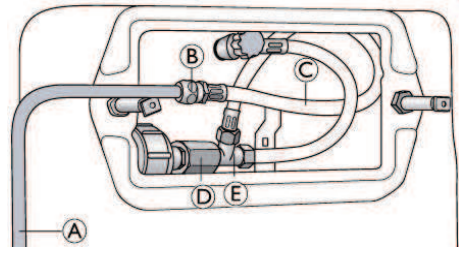


- Remove the push plate from the flush-mounted housing.
- Remove the rear cover plate to obtain a direct perspective on the connection.
- Turn the water tap off.
- Remove connecting sleeve ④ and replace by T-piece ⑤.
- Connect T-piece with PVC Flexible hose ③.
- Connect PVC Flexible hose with tube ① using nut ②.
- Connect tube ① with pure bidet.
- Turn the water tap on.

NOTICE!

— Check the connection for leaks and re-fit if necessary.

-

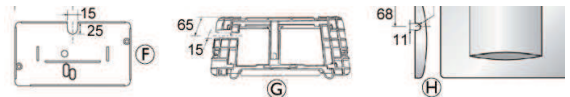Cut-out inner cover plate ⑥, mounting plate ⑦ and push plate ⑧ according to the figure above taking into account the indicated dimensions [mm].

- Re-attach the inner cover plate and the mounting plate inside the flush-mounted housing.
- Close flush-mounted housing with the push plate.


Following parts are needed for the assembly of the flush-mounted water flow:

- Tube ①
- Nut ②

Available at your specialised provider.

4.3 Trial Operation

Leakage Test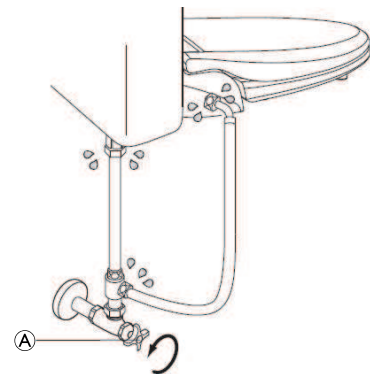


- Open the shut off valve ① completely by turning counterclockwise.
- Check for any water leakage on the connection hoses, tighten if necessary.

Connecting the Power Plug

- Read and follow the safety information regarding electrical safety, → 2.2 Safety Information for Electrical System, 3.
- Connect power plug to a suitable power source.
- If the power indication light is not turned on after the power plug connection, press any button except HI/LO, FR/BK buttons of the remote control, then power goes on.

Checking the Key Functions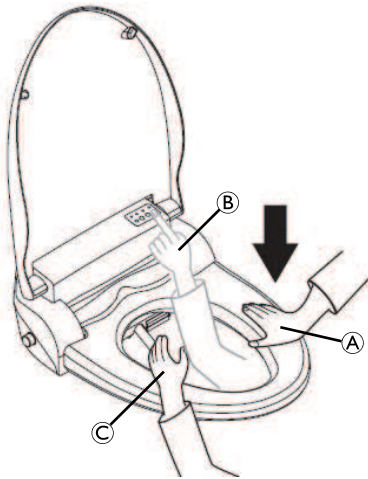


1. Place one hand on the seat sensor (A).
2. With the other hand press "CLEANSING / STOP" on the unit control panel (B) or "CLEANSING" or "BIDET" on the remote control.
 - Water sprays out of the nozzle. Cover the nozzle with your free hand (C) to prevent water spattering out.

4.4 Disassembly


Follow the below instructions if in need of detaching the unit for removal.

1. Disconnect the power plug.
2. Close the water supply by turning the shut off valve clockwise.
3. Drain the toilet water tank completely by flushing.
4. Disconnect the PVC flexible hose from the T-valve.
- 5.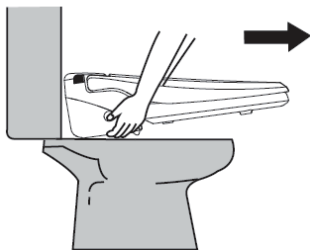

Lift the device slightly and pull it forwards out of the holder.

6.

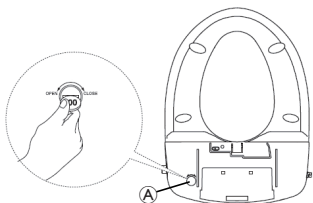Unfasten the drain plug (A) underneath the unit and drain the water completely out of the water tank.

7. Disconnect the PVC flexible hose and T-valve.
8. Connect the water-supply hose from the water tank to the shut off valve.
9. Open the shut off valve by turning counterclockwise.
10. Check the connections for any leakages, tighten if necessary.

4.5 Changing Batteries of Remote Control

The battery life cycle depends on the frequency of use.

CAUTION!

Risk of Remote Control Damage

- Only use batteries of type AA (LR6) 1.5 V.
- Insert batteries according to the +/- indicator in the remote control.
- Do not use old and new batteries together.

1. Change used batteries into new ones when "BATTERY SIGN" LED blinks on the remote control, see section *Assembling the Remote Control* in 4.1 *Installation / Assembly*, 5.

4.6 Transport and Storage

NOTICE!

- If not in use for a long time, drain the water tank completely, see 4.4 *Disassembly*, 7.

NOTICE!

Risk of Damage to the Product

- Store the product indoors.
- Store and transport the product in a dry environment. For temperature and humidity limits see 9.3 *Environmental Parameters*, 12.
- Do not store the product near heat sources.
- Keep the product out of direct sunlight.

5 Usage

5.1 Safety Information

CAUTION!

Risk of Injury or Damage to Property

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.

Make sure that the settings you have chosen fit to your state of health. Always adapt the water / air dryer temperature, the pressure and frequency of use to your needs.


5.2 Functional Description of Self-cleaning Mechanism

The self-cleaning mechanism of the showering and cleansing nozzle is automatically activated before and after each function / use.


The showering and cleansing nozzle is automatically retracted and rinsed with fresh water for approx. 5 seconds. Since the shower and cleansing nozzle is at this time within the device, water cannot be splashed when the lid is open. When pressing a function key, the nozzle moves out again and is ready for use.

When the device has been in standby-mode or has been re-connected to the power supply, the self-cleaning mechanism is activated once.

5.3 Remote Control and Control Panel Overview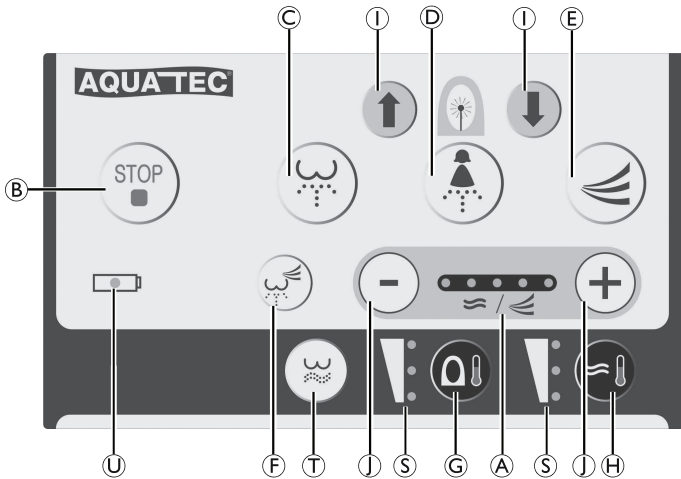

Overview of Key Functions on Remote Control		
Pos.	Key Name	Function
Ⓐ	WATER / DRY	Display of the current water pressure or the temperature of the warm air dryer
Ⓑ	STOP (red button)	Red Button: Stops all active functions
Ⓒ	CLEANSING/MOVE	Cleansing function is performed
Ⓓ	BIDET/MOVE	Bidet function is performed
Ⓔ	DRY	Warm air dryer function begins to operate
Ⓕ	AUTO	Automatic cleaning function starts
Ⓖ	SEAT	Adjusting seat temperature
Ⓗ	WATER	Adjusting water temperature
Ⓘ	FRONT / BACK	Change nozzle position
Ⓙ	HIGH / LOW	Adjusting warm air dryer temperature Adjusting water pressure
Ⓣ	HIP BATH	Hip bath cleansing is performed
Ⓢ	HIGH / LOW	Display of current water temperature or seat temperature
Ⓤ	BATTERY SIGN	Illuminates when batteries need to be replaced

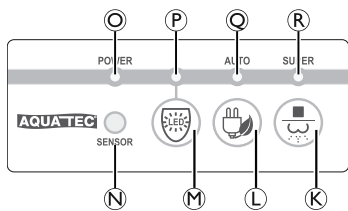Overview of Key Functions on Control Panel		
Pos.	Key Name	Function
Ⓚ	CLEANSING/STOP	Device turns off
Ⓛ	POWER SAVING	Power-saving mode activates
Ⓜ	LIGHT	Night-light activates
Ⓝ	LIGHT SENSOR	When night-light function is activated, bidet automatically senses toilet light and night-light LED switches on
LED Lights Indicate the Mode of the Device:		
Ⓞ	POWER	Unit switched on
Ⓟ	LIGHT	Night-light activated

Overview of Key Functions on Control Panel		
Pos.	Key Name	Function
Ⓞ	AUTO	Automatic power-saving mode activated
Ⓡ	SUPER	Super power-saving mode activated

5.4 Using the Shower Toilet

Perform Manual Cleansing

A sensor is located inside the toilet seat, that is only activated by skin contact.

- If plugged-in, the device is turned on and the "POWER" LED Ⓞ illuminates on the control panel.
 - If the "POWER" LED does not illuminate, press any button on the remote control (except HIGH/LOW or FRONT/BACK buttons).
- Set yourself centered on the toilet seat.
- Press the **CLEANSING/MOVE** button Ⓒ or **BIDET/MOVE** button Ⓓ.
 - CLEANSING respectively BIDET-function is performed.
- Press again **CLEANSING/MOVE** button or **BIDET/MOVE** button during the cleansing process.
 - The cleansing nozzle moves back and forth. Nozzle stops by pressing the button again.
- Press **STOP** button Ⓑ on the remote control.
 - Shower procedure stops.
- Press **DRY** button Ⓔ on the remote control.
 - Warm air dryer is activated. Dryer function stops by pressing the "STOP" button.
- Press **CLEANSING/STOP** button Ⓚ and hold for more than 4 seconds.
 - The device is turned off — All displays switch off on the control panel.

CLEANSING/MOVE and BIDET/MOVE function operates for 90 seconds. Hip bath function operates for 3 minutes. The functions are ending with an audible beep. Before and after each operation, the self nozzle cleaning function runs and sanitizes the nozzle head.

Perform Automatic Cleansing

The automatic cleansing function is a default function in which a complete cleansing cycle is automatically executed.

- Press **AUTO** button Ⓕ on the remote control.
 - The 3rd level for water pressure, the 2nd level for warm air dryer temperature and the last setting of nozzle position are automatically set-up.
 - The Sensor forces to set the water- and seat temperature to "Medium", in case the temperature were set up on the "High" position.
 - A buzzer sounds when the auto cleansing function finishes.
- Press **STOP** button Ⓑ on the remote control.
 - The automatic cleansing process is interrupted.
- Press **AUTO** button Ⓕ again.
 - The previous automatic cleansing process is resumed.

Automatic cleansing function: Move cleansing and warm air dryer run for 1 minute each.

Setting Water Pressure

The water pressure can be set in three steps (indicated by first, middle and last LED):

1. Press CLEANSING/MOVE button  on the remote control.
2. Press HIGH respectively LOW button  on the remote control.
 - The water pressure of the cleaning nozzle is raised or lowered by one step and displayed on the LED .

 The last set value of the water pressure is automatically saved by the device.


Setting Nozzle Position

1. Press FRONT respectively BACK button  during cleansing function on the remote control.
 - The cleansing nozzles move forwards respectively backwards.

 When the cleansing process is complete and you move away from the seat, the nozzle moves back into the last setting of the nozzle position.

Setting Water Temperature

The water temperature can be set in four steps:

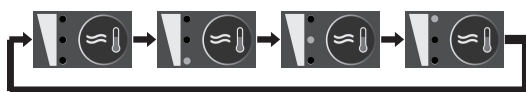1. Press WATER button  repeatedly on the remote control, until the desired temperature is reached.
 - The water temperature is displayed on the LED display above.

 The last value of the water temperature is automatically saved by the device.

Setting Seat Temperature

The seat temperature can be set in four steps:

1. Press SEAT button  repeatedly on the remote control, until the desired temperature is reached.
 - The seat temperature is displayed on the LED display above.

 The last value of the seat temperature is automatically saved by the device.

The seat is equipped with an internal combustion protection system. If the seat temperature is set to "High" and you remain longer than 30 minutes on the seat, the seat temperature drops automatically. As soon as you leave the seat, the temperature returns to the setpoint.

Setting Warm Air Temperature

The warm air temperature can be set in five steps:

1. Press DRY button  on the remote control.
2. Press HIGH respectively LOW button  on the remote control.
 - The temperature of the warm air dryer is raised or lowered by one step and displayed on the LED .

 The last value of the warm air dryer temperature is automatically saved by the device.

Turning On/Off Night-light LED


1. Press LIGHT button  on the units control panel.
 - The night-light is activated when it gets dark in the room.
 -  When the night light is activated, the device automatically recognizes the toilet light and the night light is turned off. The user can conveniently bidet without toilet light. The power consumption of the night-light is very low.
2. Press LIGHT button again to switch off the night-light.

Automatic Power Saving Mode

1. Press POWER SAVING button  repeatedly, until "AUTO" LED  illuminates.
 - Seat temperature is automatically set at 30 °C. Auto power saving function is temporarily released, when sitting on the seat. Auto power saving function runs again, when moving away from the seat.
2. Press POWER SAVING button twice.
 - Power saving mode deactivates and "AUTO" LED turns off.

Super Power Saving Mode

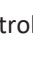Bidet senses toilet luminosity for itself and shows excellent power saving function.

1. Press POWER SAVING button  repeatedly, until "SUPER" LED  illuminates.
 - Water and seat temperature maintain normal temperature.
 -  Power saving mode is temporarily released when toilet light turns on. It returns to normal mode from power saving mode even if occupying the seat in dark condition.
2. Press POWER SAVING button once.
 - Power saving mode deactivates and "SUPER" LED turns off.

Perform Hip Bath Function

1. Press HIP BATH button  on the remote control.
 -  The hip bath function operates for 3 minutes (The nozzle moves slightly forwards and backwards). The lowest level of water pressure and the highest level of water temperature is pre-set.
2. Press HIP BATH button again to stop the nozzle moving forwards and backwards.
3. Press STOP button to cancel the hip bath function.

6 Maintenance and Servicing

If the cleaning and safety notices are observed, the product will be maintenance free.

WARNING!

- Check the product regularly for damage and ensure that it is securely assembled.
- In the event of damage and/or necessary repairs, contact your provider immediately.
- The product should be checked by a qualified technician before every re-use or after 24 months at the latest.

6.1 Cleaning and Disinfection

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

! NOTICE!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking into use again.

 For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

6.1.1 Cleaning Intervals

! NOTICE!

- Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product:
- regularly while in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

6.1.2 Cleaning Instructions

DANGER!


Electric shock hazard

- Observe safety information for electrical safety, see 2.1 *General Safety Information*, 2.
- Do not use high pressure or steam cleaners / autoclave.
- Do not touch the power socket with wet hands.
- Make sure that no water splashes to the plug or the wall outlet.

Cleaning the Shower Toilet

1. Disconnect the power plug from the wall outlet before cleaning the unit.
2. Clean the unit with moist, well-wrung soft cloth. For excessive stains, use soft cloth moistened with mild detergent solution.
3. Wipe out the device with a dry cloth.


Cleaning the Water Strainer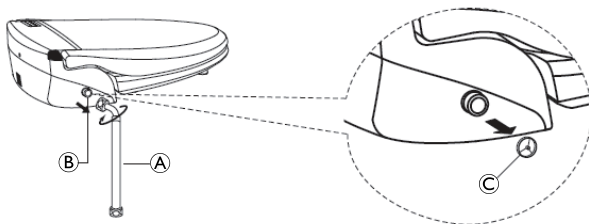

1. Close the water supply by turning the shut off valve clockwise.
2. Detach the PVC flexible hose **A** from the inlet adapter **B**.
3. Detach the strainer **C** from the inlet adapter with a pointed object.
4. Thoroughly clean the strainer with a soft brush.
5. Reinsert the strainer on the inlet adapter.

6. Attach the PVC flexible hose to the inlet adapter.
7. Open shut off valve fully by turning counterclockwise.

Cleaning the Nozzles

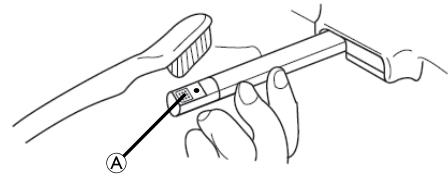1. Press “CLEANSING / STOP” and “POWER SAVING” buttons simultaneously on the control panel.
 - The nozzle **A** comes out and stops.
2. Clean the nozzle manually with a soft brush.
3. Press “CLEANSING / STOP” button.
 - The nozzle returns to its storage position and the self cleaning function is automatically activated.

6.1.3 Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.
 Disinfectant: Ordinary household disinfectant.
 Drying: Allow the product to air-dry.

7 After Use

7.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 6 *Maintenance and Servicing*, 9.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7.2 Disposal

WARNING!

Environmental Hazard

- Device contains batteries. This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.
- DO NOT dispose of batteries in normal household waste.
 - DO NOT throw batteries into a fire.
 - Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
 - For information on the battery type see battery label or 9.1 *General Data*, 11.

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

8 Troubleshooting

Fault	Possible Cause(s)	Remedy
Unit does not operate at all	Power is off. Power plug is not connected. There is a power failure	Press any button to check except "HIGH", "LOW", "FRONT" or "BACK" buttons of the remote control.
CLEANSING, BIDET, DRY, AUTO do not operate	Seat is covered.	Make sure that there is nothing on the seat surface. (The device is only activated when the user's skin touches the seat sensor.)
Remote control does not operate at all	Batteries in the remote control have been exhausted.	Replace batteries with new ones, → 4.1 <i>Installation / Assembly</i> , 5.
Malfunction of the remote control	Batteries are empty or too weak. Remote control out of reach of the device.	Replace batteries with new ones, → 4.1 <i>Installation / Assembly</i> , 5.
No water spray	The shut off valve is closed.	Open shut off valve.
	There is water supply outage.	Restore water supply.
Low water pressure	The water pressure level is set to "LOW".	Increase water pressure level, → 5.4 <i>Using the Shower Toilet</i> , 8.
	The water strainer is clogged.	Clean water strainer, → 6.1.2 <i>Cleaning Instructions</i> , 10.
	Nozzle head is dirty.	Clean nozzle, → 6.1.2 <i>Cleaning Instructions</i> , 10.
Low water temperature	The water temperature level is turned off or set to "LOW".	Turn on or increase water temperature level, → 5.4 <i>Using the Shower Toilet</i> , 8.
Seat is not warm	The warm seat temperature level is turned off or set to "LOW".	Turn on or increase seat temperature level, → 5.4 <i>Using the Shower Toilet</i> , 8.
Dryer air is not warm	The warm air dryer temperature level is turned off or set to "LOW".	Turn on or increase warm air dryer temperature level, → 5.4 <i>Using the Shower Toilet</i> , 8.

9 Technical Data

9.1 General Data

Power supply	220 – 240 V AC, 50/60 Hz
Max. power consumption	1725 – 2053 W

Intermittent (periodic operation)	85%, 30 min. ON / 5 min. OFF
Type of protection, entire device	IPX4 1)
Length of power cord	1.4 m
Battery type remote control	2 x AA (LR6) / 1.5 V
Type water supply connection	Direct connection to the water supply
Ø Connection at shower toilet	1/2"
Ø T-piece	1/2" - 3/8" - 1/2"
Ø Connection hose	1/2" cranked to 3/8"
Length connection hose	500 mm
Water supply pressure	79 – 735 kPa
Water supply temperature	3 – 25 °C
Operating temperature	3 – 40 °C
Product width	389 mm
Product length	515 mm
Product depth	156 mm
Product weight	3.9 kg
Maximum user weight	125 kg

1) Protected against the penetration of water that is splashed against the housing from all directions (warm seat only).

Cleansing Functions	
Water flow rate CLEANSING	0.7 l/min
Water flow rate BIDET	0.85 l/min
Water pressure adjustment	3 levels
Temperature adjustment	Normal water supply temperature, 34 °C, 37 °C, 40 °C
Heater capacity	1670 – 1987 W
Safety devices	Thermal fuse, Thermostat

Warm Seat	
Temperature adjustment	Room temperature, 35 °C, 38 °C, 40 °C
Heater capacity	55 – 65 W
Safety devices	Thermal fuse

Drying Functions	
Temperature adjustment	Room temperature – 50 °C (5 levels)
Heater capacity	195 – 232 W
Safety devices	Thermal fuse, Bi-metal

9.2 Materials

Seat / Lid	PP
------------	----

All product components are corrosion resistant and do not contain natural rubber latex.

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- S'il est branché, l'appareil est allumé et la LED « POWER » (MISE SOUS TENSION) © du panneau de commande est allumée.
Invacare AG
Invacare AB
Invacare GmbH